

# Use the **VUR Index** to Help Create Your Patient's VUR Management Plan<sup>1</sup>

The Vesicoureteral Reflux Index (VURx) can predict primary VUR improvement or resolution in children.<sup>2\*</sup>


While VUR grade plays an important role in assessing the severity of the VUR, the VURx accounts for other important factors which can help you determine the best management plan for the patient.

VURx is more predictive of breakthrough UTIs than international reflux grade alone.<sup>1</sup>


**Use the VURx scoring chart to the right to calculate the probability of spontaneous resolution and the risk of future or recurrent fUTI, referenced in the charts below.**

VCUG DATA	SCORE
<b>Timing of Reflux</b> 3 Filling (Early-Mid) 2 Filling (Late) 1 Voiding (Only)	<input type="text"/> +
<b>Ureteral Anomalies</b> (Duplex/PUD) 1 Present 0 Absent	<input type="text"/> +
<b>VUR Grade</b> 1 High (4-5) 0 Low-Mod (1-3)	<input type="text"/> +
<b>Gender</b> 1 Female 0 Male	<input type="text"/> ↓
<b>TOTAL</b>	<input type="text"/> out of 6

**Probability of spontaneous resolution<sup>3\*</sup>**

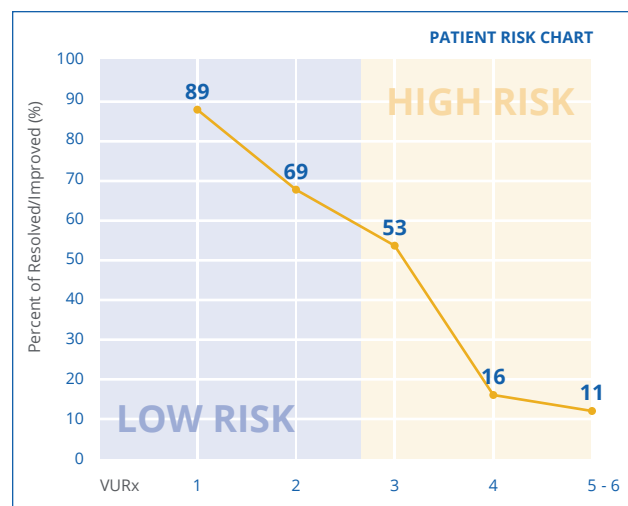

**Risk of future or recurrent fUTI<sup>1</sup>**

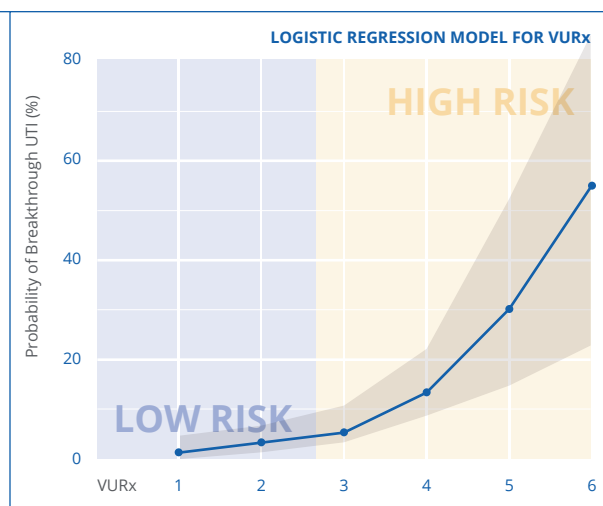

\*Resolution is less likely as VURx score and time from diagnosis increase for children over 2 years of age. Resolution is defined as no VUR or 2 or more grade improvement.

## References

- Arlen AM, Leong T, Wu CQ, Traore EJ, Cooper CS, Kirsch AJ. Predicting breakthrough UTI: Comparative analysis of VUR index (VURx), VUR grade and ureteral diameter ratio (UDR). *J Urol.* 2020;204:1-6.
- Kirsch AJ, Arlen AM, Leong T, et al. Vesicoureteral reflux index (VURx): A novel tool to predict primary reflux improvement and resolution in children less than two years of age. *J Urol.* 2014; 10(6):1249-54.
- Garcia-Roig, M, Ridley DE, McCracken C, Arlen AM, Cooper CS, Kirsch AJ. Vesicoureteral reflux index: predicting primary vesicoureteral reflux resolution in children diagnosed after age 24 months. *J Urol.* 2017;197:1150-1157.

**U.S. CAUTION:** Federal (USA) law restricts this device to sale by or on the order of a physician.

Deflux, and the Deflux logo are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. Rx Only.

© 2025 Teleflex Incorporated. All rights reserved. **APM1210A**

## Medical Information Department

For product complaints and adverse event reports, contact our Medical Information Department:

**Phone:** 844-350-9656  
**Fax:** 510-595-8183  
**Email:** [palettemc@eversana.com](mailto:palettemc@eversana.com)

For more information about Deflux™ injectable gel, please visit [deflux.com](http://deflux.com).