## Deflux® Instructions for Use

### Composition

Each ml contains:  
Dextranomer 50 mg  
Hyaluronic acid, stabilized 15 mg  
Phys. sodium chloride solution q.s.

### Description

Deflux is a sterile, viscous gel of debranomer microspheres and stabilized hyaluronic acid for intravesical instillation, a biocompatible and biodegradable implant. Deflux is intended for submucosal injection in the urethra, bladder and distal ureter. The stabilized hyaluronic acid may act as a carrier, leaving the debranomer microspheres at the implant site where they gradually surround by tissue. The stabilized Deflux is supplied in a glass syringe with a luer-lock fitting containing 1 ml. Each syringe is terminally moist sterilized in a pouch and packed in a paper carton. The product is for single use only. The syringes shall be furnished with indicative volume markings at 0.1 ml intervals. Patient record labels are included in the package to facilitate traceability of the product.

### Indications

- Stress urinary incontinence due to urethral sphincter incompetence. Preliminary clinical data suggest that Deflux may also be effective for stress urinary incontinence due to urethral hypermobility.
- Vesicoureteral reflux.

### Contraindications to treatment of vesicoureteral reflux

- Primary refluxing neophytes with distal stenosis.
- Uncontrolled voiding dysfunction.
- Warnings**
  - Deflux is only intended for submucosal injections in the urethra, urinary bladder and distal ureter.
  - Do not resterile Deflux as this will damage the product.
  - Do not inject intravascularly.
  - Do not mix with other products.
  - Do not inject if the patient is known to be allergic to hyaluronic acid-based products or dextran.

\* The procedure and instrumentation associated with the injection of Deflux carry an inherent risk of urinary tract infection or bleeding as do similar urological procedures. The usual precautions associated with urological procedures performed under sterile conditions, specifically cystoscopy, should be observed.

\* Do not inject more than 15 ml of Deflux in adults and 6 ml in children at the time of the first treatment session.

\* Inject no more than 1 ml of water on the luer-lock connection which could cause leakage of the gel.

\* Should care be taken with the handling of the glass syringe and needle to avoid laceration or other injury.

\* In the event of accidental contamination of the device it should be discarded.

\* The safety and effectiveness of Deflux in pregnant or lactating women has not been established.

\* Do not use the product if the package is damaged.

Interaction with Deflux in combination with drugs and other medical devices has not been studied.

### Adverse Events

#### Stress Urinary Incontinence

Common procedure-side effects are dysuria, urinary tract infection, hematuria, and pain at the injection site. These usually resolve spontaneously within a few days as with any appropriate antibiotic treatment. Postoperative urinary retention generally requires catheterization by a clinician. If catheterization is required it should occur within 20% of patients. If incontinence is persistent it is recommended that a urologist (10 Fr) catheter with generic lubrication is used, so as to reduce the risk of compromising the implant site. Prolonged catheterization is necessary suprapubic cystostomy is recommended for the same reason.

Injection site reactions such as mass/bacillus/pseudocyst have been reported.

#### Vesicoureteral Reflux

Postoperative dilation of the upper urinary tract that resolves spontaneously within a few days has been observed in clinical investigations in less than 1% of treated patients.

However, in cases of incomplete dilation of the upper urinary tract when evr or without hyaluronic acid leading to temporary placement of a ureteric stent have been reported. In very rare cases of ureteral obstruction, ureteric reimplantation has been required.

Incidental findings of injection site calcifications which have been missed for distal stenosis can be possible diagnosis confounder in imaging studies. Future physicians should be informed that a patient had a treatment with Deflux. Adverse events should, however, be related to the product should be reported to the manufacturer or local retailer.

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